



610.756.7863

[Home](#) » [Products](#) » [Ayers Chalazion Forceps](#)

AYERS CHALAZION FORCEPS

Product Number: 66-1435

3-1/2" Ayers Chalazion Forceps

Sklar® Ayers Chalazion Forceps are ophthalmologic thumb forceps used to evert the eyelid and expose chalazion for drainage and removal. The instrument has two circular tips. One is solid and the other is fenestrated. The chalazion on the eyelid is isolated between the two tips and the forceps are tightened by an attached screw, which pushes the chalazion upwards so a procedure can be completed. The size and consistency of the chalazion are primary factors when choosing which chalazion forceps to use. Larger chalazion, as well as those with hard matter, will require larger incisions. Therefore, forceps with larger tips may be more useful. Similarly, smaller chalazion, or those determined to contain only fluid, may be excised with smaller incisions and the use of forceps with a smaller diameter. This product is straight with smooth tips and a length of 3-1/2 inches. The inside diameter of the fenestrated ring is approximately 8mm.



Additional Information

Surname	Ayers
Length	3-1/2"
Tip Configuration	1 Side Fenestrated
Instrument Type	Ophthalmic Forceps
Curvature	Straight
Working Surface Style	Smooth
Material	Stainless Steel
Disposable or Reusable	Reusable
Sterile or Non-Sterile	Non-Sterile
Latex or Latex-Free	Latex-Free
Grade	Premium OR-Grade

© 2016 Sklar Surgical Instruments. All Rights Reserved.

889 South Matlack Street, West Chester, PA 19382 • Phone: 610.756.7863 • Fax: 610.430.3941 • Service@SklarCorp.com